



Contents

Introduction	4
Univox® CLS-1 loop driver	5
CTC-120 with glass/wall microphone – installation guide	6
CTC-121 with goose neck microphone – installation guide	8
Troubleshooting.....	10
Measuring devices.....	10
Univox® Listener	
Univox® FSM, field strength meter	
Safety and warranty.....	11
Maintenance and care	11
Service	11
Trouble shooting.....	11
Environment	11

Introduction

CTC Cross-The-Counter systems are complete systems for equipping reception desks and counters with an induction loop. The system consists of a loop driver, a loop pad, a microphone and a wall holder. Installed in a reception desk or counter, the system gives hearing hearing aids users the possibility to communicate with the staff behind the desk with greatly enhanced speech perception.

The system is always activated and no special preparations have to be undertaken, neither by the hearing impaired nor by the staff. The only requirement for the hearing aids user is to put their hearing aids in T-position and for the staff to speak normally into the microphone.

All Univox® drivers have a very high output current capability resulting in powerful and secure products fulfilling existing standards, IEC 60118-4.

Thank you for having chosen a Univox® product.

Univox CTC-120

Univox CLS-1 loop driver

Univox 13V microphone for glass/wall

Loop pad, Sign/label with T-symbol 80 x 73 mm

Wall holder for loop driver

Part No: 202040A (EU) 202040A-UK 202040A-US 202040A-AUS

Univox CTC-121

Univox CLS-1 loop driver



Univox M-2 goose neck microphone

Loop pad, Sign/label with T-symbol 80 x 73 mm

Wall holder for loop driver

Part No: 202040B (EU) 202040B-UK 202040B-US 202040B-AUS

Univox® Compact Loop System CLS-1

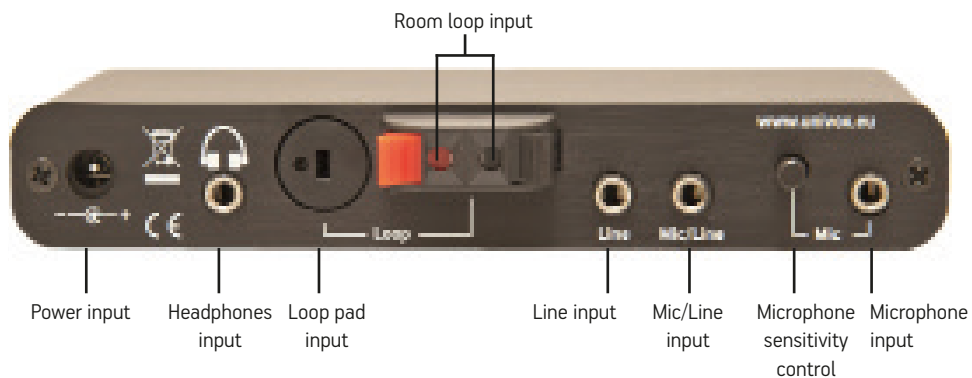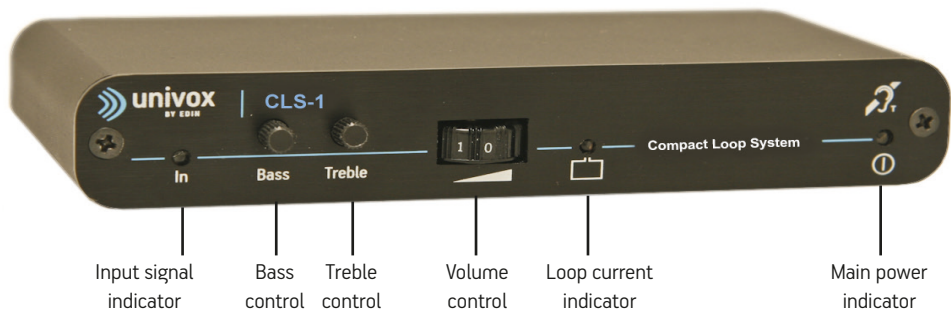

T-symbol label



Wall holder for loop driver



Loop pad



AVLM5 microphone for glass or wall




M-2 gooseneck microphone


Installation guide for CTC-120

with microphone for glass or wall

Installation and commissioning

1. Choose a suitable place for the loop driver. Consider that the loop pad, the microphone and the loop driver's power supply will be connected to the driver. If needed, attach the wall holder facing upwards on the selected spot.
2. Choose a suitable location for the microphone. It can be placed on a wall or on glass. When choosing a place for the microphone, consider that the staff shall be able to stand or sit and talk in a normal, relaxed way with the listener. An example of how the system can be laid out, see fig. 1. Place the microphone cable under the desk in such a way that it will reach to where the loop driver/wall holder is mounted. The microphone cable is 1.8 meters.
3. Mount the loop pad under the reception desk. The loop pad should be attached in the angle between the front and the upper part of the reception desk as shown in fig.1 and 2. This will ensure a constant field distribution with the right direction and also allow hearing aid users to tilt their head forwards, for example when writing. When mounting the pad (be careful not to damage the loop cables inside the pad), place the loop pad cable in such a way that it will reach the loop driver/wall holder. The loop pad cable is 10 meters.

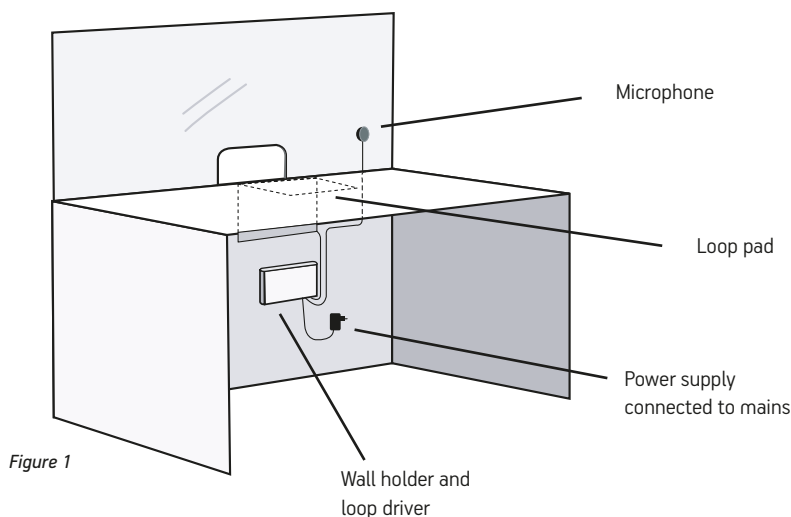

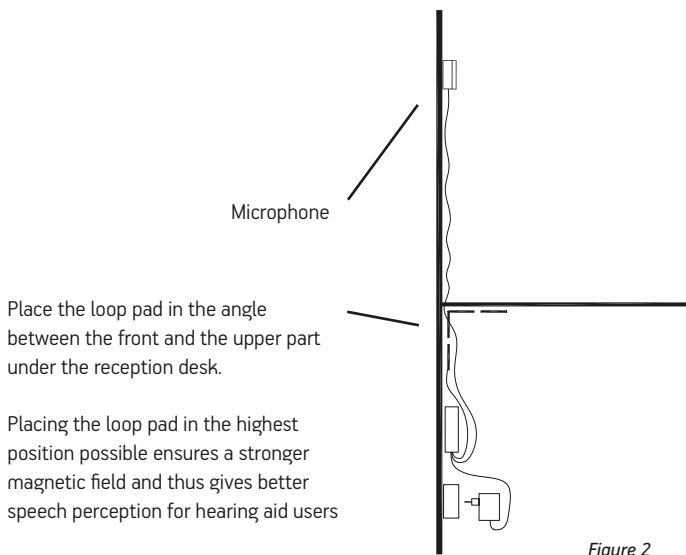

4. Connect the cables power supply, loop pad and microphone, see page 5. If the wall holder is being used, run the cables from the loop driver's power supply, loop pad and microphone through the wall holder from underneath. Place the driver in such a way that the connector side is facing down and you can read the text on the front of the driver in the right direction. Connect all three cables, see page 5. Finally, lower the driver into the wall holder and connect the power supply to mains.
5. When all connections are completed correctly the LED-indicator for mains power on the right hand side of the front of the driver shall light up. The system is now ready to be used.
6. The loop current is adjusted by turning the volume control at the front of the driver. Verify the loop level/volume with a Univox® Listener. Bass and treble controls shall only be adjusted in exceptional cases.


Installation guide CTC-121

with gooseneck microphone

The system is always activated and no special preparations have to be undertaken, neither by the hearing impaired nor by the staff. The only requirement for hard of hearing people is to put their hearing aids in T-position and for the staff to speak normally into the microphone.

Installation and commissioning

1. Choose a suitable place for the loop driver. Consider that the loop pad, the microphone and the loop driver's power supply shall be connected to the driver. If needed, attach the wall holder facing upwards on the selected spot.
2. Choose a suitable place for the microphone. It can be placed on a desk or a table. When choosing the place for the microphone, consider that the staff shall be able to stand or sit and talk in a normal, relaxed way with the listener. An example of how the system can be laid out, see Pic. 3. Place the microphone cable under the desk in such a way that it will reach the place where the loop driver/wall holder is mounted. The microphone cable is 1.5 meters.
3. Mount the loop pad under the reception desk. The loop pad should be attached in the angle between the front and the upper part of the reception desk as shown in fig. 3 and
4. This will ensure a constant field distribution with the right direction and also allow

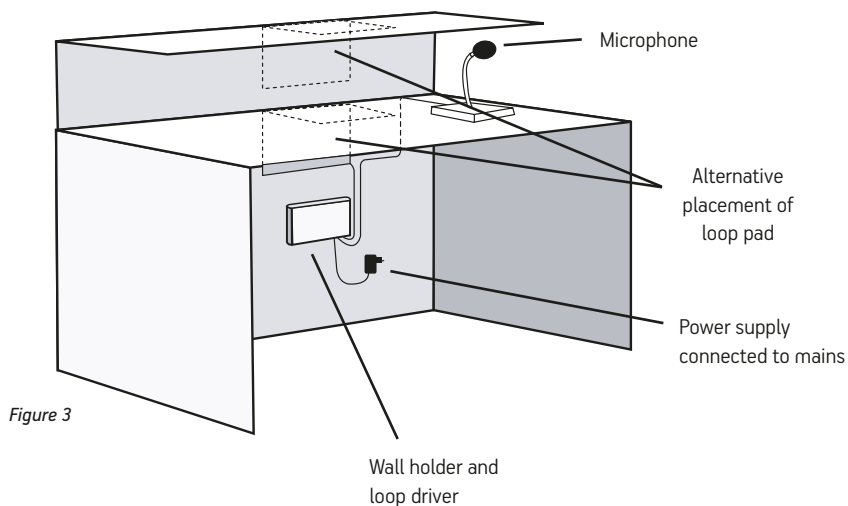

The loop pad can be placed either under the top of the desk or under the top of the reception table.

Placing the loop pad in the highest position possible ensures a stronger magnetic field and thus gives better speech perception for hearing aid users

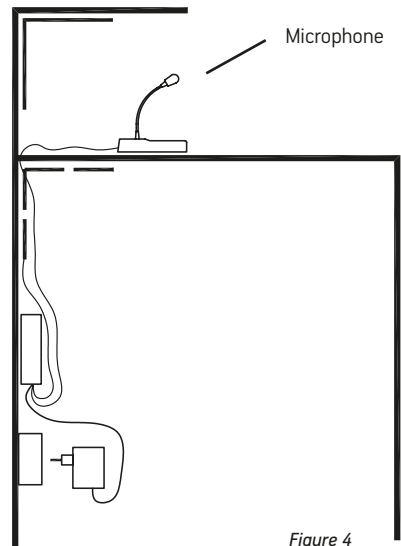


Figure 4

hearing aid users to tilt their head forwards, for example when writing. When mounting the pad (be careful not to damage the loop cables inside the pad), place the loop pad cable in such a way that it will reach the loop driver/wall holder. The loop pad cable is 10 meters.

4. Connect the cables power supply, loop pad and microphone, see page 5. If the wall holder is being used, run the cables from the loop driver's power supply, loop pad and microphone through the wall holder from underneath. Place the driver in such a way that the connector side is facing down and you can read the text on the front of the driver in the right direction. Connect all three cables, see page 5. Finally, lower the driver into the wall holder and connect the power supply to mains.
5. When all connections are completed correctly the LED-indicator for mains power on the right hand side of the front of the driver shall light up. The system is now ready to be used.
6. The loop current is adjusted by turning the volume control at the front of the driver. Verify the loop level/volume with a Univox® Listener. Bass and treble controls shall only be adjusted in exceptional cases.

Troubleshooting

Verify the control LEDs following the instructions in this installation guide. Use Univox® Listener to check the sound quality and basic level of the loop. If the loop driver does not perform satisfactory, check the following:

- Does the mains power indicator light? If not, make sure that the transformer is correctly connected to the power outlet and to the driver.
- Is the loop current indicator lit? This is a guarantee that the system works. If not, check that the loop pad is not broken and correctly connected, and make sure to check all other connections.
- Attention! If headphones are connected the loop current indicator is disabled.
- The loop current indicator lights but there is no sound in the hearing aid/headphones: check that the M-T-O switch of the hearing aid is in T or MT mode. Also check the status of your hearing aid batteries.
- Bad sound quality? Adjust the loop current, bass and treble controls. Bass and treble adjustment should normally not be needed.

Make sure that Listener is turned on (red LED flashes). If not, change the batteries. Please make sure that the batteries are inserted correctly. If the loop receiver sound is weak, make sure that Listener is hanging/hold in vertical position. Adjust volume if necessary. Weak signal might indicate that the loop system doesn't comply with the international standard IEC 60118-4.

Should the system not work after having made the product test as described above, please contact your local distributor for further instructions.

Measuring devices

Univox® FSM Basic, Field Strength Meter

Instrument for professional measurement and control of loop systems according to IEC 60118-4.



Univox® FSM Basic

Univox® Listener

Loop receiver for fast and simple check of the sound quality and basic level control of the loop.



Univox® Listener

Safety and warranty

Basic knowledge in audio and video installation techniques is required to achieve existing regulations. The installer is responsible for the installation hereby avoiding any risk or cause of fire. Please note that the warranty is not valid for any damage or defects on the product due to incorrect or incautious installation, usage or maintenance.

Bo Edin AB shall not be held responsible or liable for interference to radio or TV equipment, and/or to any direct, incidental or consequential damages or losses to any person or entity, if the equipment has been installed by unqualified personnel and/or if installation instructions stated in the product Installation Guide have not been strictly followed.

Maintenance and care

Under normal circumstances Univox® loop drivers do not need any special maintenance. Should the unit become dirty, wipe it with a clean damp cloth. Do not use solvent or strong detergents.

Service

Should the product/system not work after having made the product test as described above, please contact the local distributor for further instructions. If the product should be sent to Bo Edin AB, please enclose a filled Service Form available at www.univox.eu/support.

Technical data

For additional information, please refer to product data sheet/brochure and CE certificate which can be downloaded at www.univox.eu/downloads. If required other technical documents can be ordered from your local distributor or from support@edin.se.

Environment

When this system is finished with, please follow existing disposal regulations. Thus if you respect these instructions you ensure human health and environmental protection.



Univox by Edin, the world's leading expert and producer of high quality hearing loop systems, created the very first true loop amplifier 1969. Ever since our mission is to serve the hearing community with the highest degree of service and performance with strong focus on Research and Development for new technical solutions.

Bo Edin AB
Deliveries

Stockby Hantverksby 3, 181 75 Lidingö
Förrådsvägen 2B, 181 41 Lidingö

Tel: 08 7671818

Email: info@edin.se
Web: www.univox.eu

Hearing excellence since 1965

