



Part no 283103

Univox
Stockby Hantverksby 3
SE-181 75 Lidingö, Sweden

+46 (0)8-767 18 18
info@edin.se
www.univox.eu

USER GUIDE

An inductive neck loop is a personal assistive listening accessory for hard of hearing people wearing hearing aids equipped with a T-coil. Each hearing aid is configured with frequency response and dynamic range tuned to the user's hearing impairment. The NL-100 transfers any audio from its connected device crystal clear, personalized and with minimal latency.

Univox® NL-100

Univox® NL-100 is capable of connecting to, and receiving signal from different sound sources. NL-100 sums both right and left channels of the audio source, ensuring a strong and steady signal for the user. Included in the package is a 70 cm (27.5 in) extension lead.

Connection and general information

How to use the NL-100:

- Wear the loop over your head around your neck
- Connect the NL-100 to the headphone jack of your device
- Turn your hearing aid to the telecoil position (T-mode)
- Adjust the volume on your device

NOTE! We recommend setting your device to minimum volume before connecting the NL-100 to avoid any sudden loud sounds that may potentially harm your hearing aid.

Maintenance, care and service

Under normal circumstances the product does not need any special maintenance. Wipe the unit with a clean damp cloth. Do not use any solvents or detergents. Periodically check cables and connectors for any damage. Should the unit not work as expected, contact your dealer or local distributor for further instructions.

Environment

To prevent possible harm to the environment and human health, please dispose of the product responsibly by following local statutory disposal regulations.

Technical specifications

Resistance		2 x 12.5 Ohm
Sensitivity		0dB @ 55mVrms
Frequency response		50Hz-15kHz, -3dB, rel. 1kHz
Connector		3.5 mm plug
Dimensions	Circumference	74 cm (29 in)
	Cable	20 cm (7.8 in)
	Extension cord	70 cm (27.5 in)
Weight (net)		20 g (0.7 oz) incl extension cord
Part No		283103

