

Univox® NL-100

Neck Loop



USER MANUAL

An inductive neck loop is a personal assistive listening accessory for people who are hard of hearing and use hearing aids equipped with a T-coil. Each hearing aid is configured with a frequency response and dynamic range tuned to the user's hearing impairment. The NL-100 delivers audio from its connected device with crystal-clear quality, personalized sound, and minimal latency.

Univox® NL-100

The Univox® NL-100 can connect to and receive signals from various audio sources. It combines the right and left audio channels to ensure a strong and consistent signal for the user. A 70 cm (27.5 in) extension lead is included in the package.

Connection and general information

How to use the NL-100:

- Wear the loop around your neck
- Connect the NL-100 to the headphone jack of your device
- Turn your hearing aid to the telecoil position (T-mode)
- Adjust the volume on your device

NOTE! We recommend setting your device to minimum volume before connecting the NL-100 to avoid sudden loud sounds that could damage your hearing aid.

Maintenance, care and service


Under normal conditions, the product does not require special maintenance. Wipe the unit with a clean, damp cloth. Do not use solvents or detergents. Periodically inspect cables and connectors for any damage. If the unit does not work as expected, contact your dealer or local distributor for further instructions.

Environment



To prevent possible harm to the environment and human health, please dispose of the product responsibly by following local disposal regulations.

TECHNICAL SPECIFICATIONS

| | | |
|------------------|----------------|--|
| Resistance | | 2 x 12.5 Ohm |
| Cutoff frequency | | 2.6 kHz, -3 dB @ 1kHz |
| Connector | | 3.5 mm plug |
| Dimensions | Circumference | 74 cm (29 in) |
| | Cable | 20 cm (7.8 in) |
| | Extension cord | 70 cm (27.5 in) |
| Weight (net) | | 20 g (0.7 oz) incl. extension cord |
| Part no. | | 283103 |
| Standards | |  WEEE |

For more information, please visit univox.eu



Univox / Bo Edin AB
Stockby Hantverksby 3
SE-181 75 Lidingö, Sweden



info@univox.eu
univox.eu



+46 8 767 18 18

